

Lens Glide

DEVICE DESCRIPTION

A disposable Lens glide is double ended plastic (Polypropylene) instruments. The narrow end is used during phacoemulsification surgical procedure whereas the broader end is used during non-phaco surgical procedures. A Lens Glide is individually packed & sealed in a pouch before sterilization.

INTENDED USE / INDICATION FOR USE

A disposable sterile Lens Glide is used for insertion of IOLs or other microsurgical instruments in the anterior chamber. It is also used for removal of cataract Lens matter from the anterior chamber during an ophthalmic surgical procedure.

CONTRAINDICATIONS

The device is contraindicated for patients with known silicon and plastic sensitivity.

HOW SUPPLIED

Lens Glides are supplied sterile, in individual pouches. 10 such pouches are enclosed within an outer printed box. The overall packing contains Lens Glides and medical literature insert, in an outer enclosing box, with external identification stickers.

CAUTIONS

Do not use the device if the sterilized package is open or damaged.

⚠ WARNING

- Madhu Instruments Pvt. Ltd does not hold any responsibility if the device is reused. Reusing the device may cause serious problems to patients as under: -
 - a. The used Lens Glide might not have enough strength.

EXPIRATION DATE

The expiration date of the product is the sterility expiration date. The product should not be used after the indicated sterility expiration date.

RETURN OF DAMAGED PRODUCT

Return the product in its original packing identified by the batch number, purchase information, your reference and reason for return. Please contact your local distributor office regarding product return/exchange.

EXPLICATION OF SYMBOL:

Symbol	Meaning
	Caution
	Manufacturer
	Date of Manufacture
	Do not re-sterilize
	Consult Instructions for use
	Keep away from sunlight
	Don't re-use
	This side up
	Do not use hook

- b. Do not use the lens glide if there are burrs on the edges.
 - c. Contagious and communicable disease such as HIV, Hepatitis, contagious disease/communicable disease/undue diseases etc.
- Only qualified medical practitioner should use the device.
 - Do not use the device if the sterilized package is open or damaged.
 - Sterility is guaranteed only when the packing is neither opened nor damaged.
 - Do not re-sterilize the device by any method. Madhu Instruments assumes no liability for devices that have been re-sterilized.
 - Do not reuse the device.
 - Read instruction before use.
 - Store at specified temperature and humidity.
 - The used Lens Glides and pouch should be disposed off in compliance with the local regulatory requirements.
 - Do not use the device after sterility expiry date.
 - For External use only.
 - Keep out of reach of children.

PRECAUTIONS

- When handling the product care should be taken to avoid damage from handling. Avoid crushing and crimping damage to the product due to application of surgical instruments such as forceps etc.

DIRECTION FOR USE

- Open the sterile Lens Glide pouch (BY AN UNSTERILE PERSON) and drop the glide on a sterile surface without touching it.
- Take the glide and wash it with saline water.
- Insert the desired end of the glide into the anterior chamber as per requirement of the surgical procedure.

REPORTING

Adverse events and/or potentially sight-threatening complications that may reasonably be regarded as product related and that were not previously expected in nature, severity or incidence must be reported to Madhu Instruments Pvt. Ltd.

DISPOSAL OF THE USED DEVICE

The used device should be disposed off in compliance with the local regulatory requirements.

Symbol	Meaning
	Batch Code
	Use by date
	Temperature Limit
	Do not use if package is damaged
	Humidity Limitation
	Sterilised using Ethylene Oxide
	Handle with care
	Maximum stacking
	Medical Device
	Catalogue Number

MADHU INSTRUMENTS PVT. LTD.

Mfg. Facility: A 260, Okhla Industrial Area Phase-I, New Delhi-110020 India.
 Regd. Add.: F-90/3D, Okhla Industrial Area Phase-I, New Delhi-110020 India
 Customer Care No.: +91-11-42701030, 42701031
 Email: customercare@madhuinstruments.com
 Website: www.madhuinstruments.com