

**Lens Removal Cannula**

**DEVICE DESCRIPTION**

This is a disposable ophthalmic cannula of different gauge and needle lengths. This device is supplied sterile in peel open pouches.

**INTENDED USE / INDICATION FOR USE**

This product is used for irrigating and lifting out the lens nucleus and large cortical debris. Anterior chamber depth is maintained by irrigation through the port when instrument is attached to a syringe or an I/A machine.

**CONTRAINDICATIONS**

The device should not be used in patients who show hypersensitivity to any of the components of the preparation.

**HOW SUPPLIED**

Lens Removal Cannula is supplied sterile, packed in a peel open pouch. Either Fifty (50) or Ten (10) such pouches are enclosed within an outer printed box. The overall packing contains Lens Removal Cannula and medical literature insert, in an outer enclosing box, with external identification stickers.

**CAUTIONS**

The device should not be used in patients who show hypersensitivity to any of the components of the preparation.

**⚠ WARNING**

- Madhu Instruments Pvt. Ltd does not hold any responsibility if the device is reused. Reusing the device may cause serious problems to patients are as under: -
  - a. The used Lens Removal Cannula tip might be blunt or has burrs. Do not use the cannula if the cannula tip is blunt or has burrs.
- Only qualified medical practitioner should use the device.

**EXPIRATION DATE**

The expiration date of the product is the sterility expiration date. The product should not be use after the indicated sterility expiration date.

**RETURN OF DAMAGED PRODUCT**

Return the product in its original packing identified by the batch number, purchase information, your reference and reason for return. Please contact your local distributor office regarding product return/exchange.

**EXPLICATION OF SYMBOL:**

Symbol	Meaning
	Caution
	Manufacturer
	Date of Manufacture
	Do not re-sterilize
	Consult Instructions for use
	Keep away from sunlight
	Don't re-use
	This side up
	Do not use hook



- Do not use the device if the sterilized package is open or damaged.
- Sterility is guaranteed only when the packing is neither opened nor damaged.
- Do not re-sterilize the device by any method. Madhu Instruments assumes no liability for devices that have been re-sterilized.
- Do not reuse the device.
- Read instruction before use.
- Store at specified temperature and humidity.
- The used Lens Removal Cannula and pouch should be disposed off in compliance with the local regulatory requirements.
- Do not use the device after sterility expiry date.
- The device is not for implant.
- Keep out of reach of children.

**PRECAUTIONS**

- Most of the hospitals have washing machines to clean the surgical instruments. If cleaning of instruments is done manually, precautions have to be taken to reduce the risk of contamination through dirty instruments. When sharp instruments are cleaned and there is a risk of puncturing the skin, the instruments should be disinfected first (e.g. with a liquid disinfectant). Always wear gloves when cleaning instruments that are contaminated.

**DIRECTION FOR USE**

- Open the sterile Lens Removal Cannula pouch (BY AN UNSTERILE PERSON) and drop the inner pouch on a sterile surface without touching it.
- Use the Lens Removal Cannula as per the intended use and correct surgical procedure.

**REPORTING**

Adverse events and/or potentially sight-threatening complications that may reasonably be regarded as product related and that were not previously expected in nature, severity or incidence must be report to Madhu Instruments Pvt. Ltd.

**DISPOSAL OF THE USED DEVICE**

The used device should be disposed off in compliance with the local regulatory requirements.

Symbol	Meaning
	Batch Code
	Use by date
	Temperature Limit
	Do not use if package is damaged
	Humidity Limitation
	Sterilised using Ethylene Oxide
	Handle with care
	Maximum stacking
	Medical Device
	Catalogue Number

**MADHU INSTRUMENTS PVT. LTD.**

Mfg. Facility: A 260, Okhla Industrial Area Phase-I, New Delhi-110020 India.  
 Regd. Add.: F-90/3D, Okhla Industrial Area Phase-I, New Delhi-110020 India  
 Customer Care No.: +91-11-42701030, 42701031  
 Email: customercare@madhuinstruments.com  
 Website: www.madhuinstruments.com