

Green Touch

Lissamine Green Strips

DEVICE DESCRIPTION

A disposable Whatman filter paper No.1 round blunt tip (impregnated with Lissamine Green dye) sandwiched between a sticker paper stick (overall length 60mm). Each strip is packed individually in pouches before sterilization.

INTENDED USE / INDICATION FOR USE

A disposable sterile ophthalmic diagnostic strip used when superficial conjunctival or corneal tissue change is suspected.

MATERIALS USED

Whatman filter paper, Sticker Paper, Lissamine Green Powder.

TARGET AGE GROUP

Can be used for all age groups.

INTENDED USER

Any of the registered medical doctor/medical professionals.

DURATION OF USE

Transient Use (Less than 60 Minutes).

STORAGE CONDITIONS

Store at specified temperature and humidity (temperature in between 10°C to 30°C and humidity in between 45% to 65%).

CONTRAINDICATIONS

The device should not be used in patients who show hypersensitivity to any of the materials used in construction of the device.

HOW SUPPLIED

Lissamine Green strips are supplied sterile, in individual pouches. Such packed pouches are enclosed within an outer printed box. The overall packing contains Lissamine Green strips and medical literature inserted in an outer enclosing box, with external identification stickers.

CAUTION / WARNING

- Do not use the product if the package is damaged or open and consult instructions for use.


return. Please contact your local distributor's office regarding product return/exchange.

REPORTING

Adverse events and/or potentially sight-threatening complications that may reasonably be regarded as product

EXPLANATION OF SYMBOL:

Symbol	Meaning
	Caution
	Manufacturer
	Date and Country of Manufacture
	Consult instructions for use
	Keep away from sunlight
	Medical Device
	Do not use hook
	This side up
	Temperature Limit
	Humidity Limitation
	Keep Out of reach of children

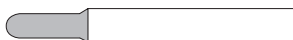

- Do not use the device if the sterilized package is open or damaged.
- The product is designed for single use only.
- If re use it may lead to cross contamination.
- The use of this product is restricted to a qualified doctor or medical practitioners.
- Keep out of reach of children.
- Do not re-sterilize the device by any method. Madhu Instruments assumes no liability for the devices that have been re-sterilized.
- Do not use the device after sterility expiry date.
- For External use only.
- Re-use of device may lead to microbial contamination due to presence of Pseudomonas.

PRECAUTIONS

- Ensure before use that product/packaging is not tampered or damaged.
- When handling the product care should be taken to avoid damage from handling.

DIRECTION FOR USE

- Check the package sealing integrity to ensure sterility.
- Remove the Lissamine Green from sterile packaging.
- Moisten Lissamine Green impregnated strip with sterile water or sterile ophthalmic solution.
- Tear fluid from the lower fornix may be used.
- While the patient looks up, touch conjunctiva or fornix with moistened tip.
- For optimal results, the patient should blink several times after application.

EXPIRATION DATE

The expiration date of the product is the sterility expiration date. The product should not be used after the indicated sterility expiration date, the shelf life for sterile products is recommended to 5 years from the date of sterilization.

RETURN OF DAMAGED PRODUCT

Return the product in its original packing identified by the batch number, purchase information, your reference and reason for

related and that were not previously expected in nature, severity or incidence must be reported to Madhu Instruments Pvt. Ltd.

DISPOSAL OF THE USED DEVICE

The used device should be disposed off in compliance applicable local, state and country's laws and regulations.

Symbol	Meaning
	Handle with care
	Maximum stacking
	Batch Code
	Use by date
	Do not use if package is damaged and consult instructions for use
	Do not re-use
	Single Sterile Barrier System, sterilized using Ethylene Oxide
	Single Sterile Barrier System with protective packaging outside, sterilized using Ethylene Oxide
	Do not re-sterilize
	Catalogue Number
	Unique device identifier

MADHU INSTRUMENTS PVT. LTD.

Mfg. Facility: A 260, Okhla Industrial Area Phase-I, New Delhi-110020 India.
 Regd. Add.: F-90/3D, Okhla Industrial Area Phase-I, New Delhi-110020 India
 Customer Care No.: 011-42701030, 42701031
 Email: customercare@madhuinstruments.com
 Website: www.madhuinstruments.com

Ref. No. MIPL/IFU/A/02, Issue No .02, Issue Date: 01.06.2015, Rev. No: 02, Rev. Date: 26.08.2023